

Legal entity of public law
L. Sakvarelidze National Center for Disease
Control and Public Health

KA030605948767920

Kakheti Highway N99, 0198, Tbilisi, Georgia; Tel: (995-32) 2192595; hot line: 116001; E-mail: ncdc@ncdc.ge

#06/2447

01/ June /2020

Mr. Papuna Papiashvili, Director
of "Elixir Global" LLC

Mr. Papuna

LEPL L. Sakvarelidze National Center for Disease Control and Public Health, processed your application regarding the registration of the disinfectant "Elixir Universal Disinfectant"

1. Information on the scope, rules and conditions of use of the disinfectant.
2. The safety data sheet
3. Conclusion on the disinfectant effect of the remedy.

Disinfectant effect of "Elixir Universal Disinfectant" was tested in the laboratory of the National Storage of Bacteria and Viruses with following strains: S.aureus ATCC 25923; P.aeruginosa ATCC 27853; B.subtilis ATCC 6633, E. hirae ATCC 10541 and B.cereus ATCC 4342.

Disinfection by exposure to annotations ensures the rendering of the above mentioned microbes harmless and can be used in accordance with the methodological recommendations.

According to the submitted documents, the main active ingredients of "Elixir Universal Disinfectant" are sodium chloride, hypochlorous acid and electrochemically activated water.

Method of use

"Elixir Universal Disinfectant" is a colorless, ready-to-use, transparent liquid with a specific odor.



The solution has a broad antimicrobial effect and washing properties. In the presence of organic contamination (blood, mucus, etc.) it is recommended to clean contaminated medical devices and remove visible contamination.

Disinfection of medical equipment is carried out by immersion in the solution, spraying and cleaning.

The solution is characterized by good washing ability and does not damage surfaces made of different materials.

The items composed of multiple parts are placed in the solution in a dismantled state, the channels and cavities should be filled with the solution during disinfection (without air plugs), and the disinfectant solution should be cover the surface not less than 2 cm above. The amount of working solution is calculated based on the volume of work and the number of individual medical items or sets, taking into account that the subject must be completely covered and placed in the disinfectant solution.

The remedy is used for hygienic hand treatment. Disinfection and pre-sterilization of surfaces of critical and semi-critical medical / dental facilities can be done in one stage, and for sterilization, should be used vessels with glass, plastic or enamel (without damage) with surface. Modes of disinfection of the surfaces of critical and semi-critical objects of medical / dental purpose and pre-sterilization and sterilization, also, the norms of solution consumption when using "Elixir Universal Disinfectant" are indicated in the methodological recommendations provided by the manufacturer.

Safety rules

Safety rules must be followed when disinfecting by "Elixir Universal Disinfectant". It is not allowed to use disinfectant after the expiration date. When using a disinfecting solution, it is necessary for the medical staff to be familiar with the label, the information about the product and to follow the manufacturer's methodical instructions carefully.

* Packaging and labeling, shelf life



Handwritten signatures and numbers, including '24' and '17123', are visible below the stamp.

customer's request. Shelf life is indicated on the label. Store in a dry, cool place away from direct sunlight.

Manufacturer

"Elixir Global" LTD, # 88 Avtomshenebeli St. Kutaisi

Tel: +995577112661

Hygienic conclusion

"Elixir Universal Disinfectant" is an effective new generation of universal means used for hygienic treatment of medical staff's hands, critical, semi-critical and non-critical group facilities, medical / dental equipment / tools and disinfection recommendations for disinfection, pre-sterilization. It is also used in educational institutions, public places and communal facilities for the hygienic treatment of hands for the disinfection of hygienic surfaces.

Regards,

Deputy General Director,

Irma Khonelidze

Signed / stamped electronically



Handwritten signature or initials in blue ink.